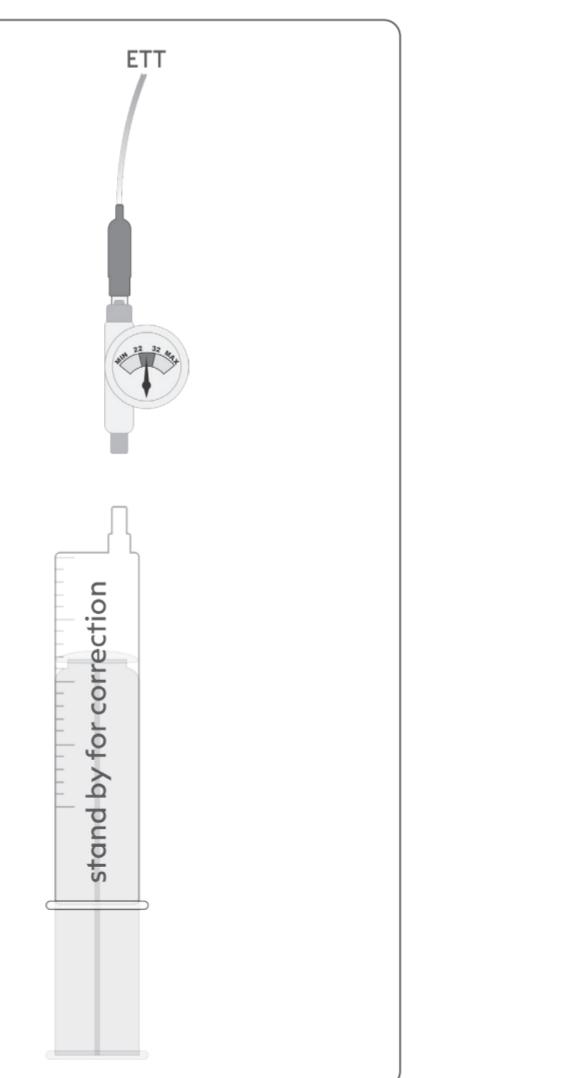
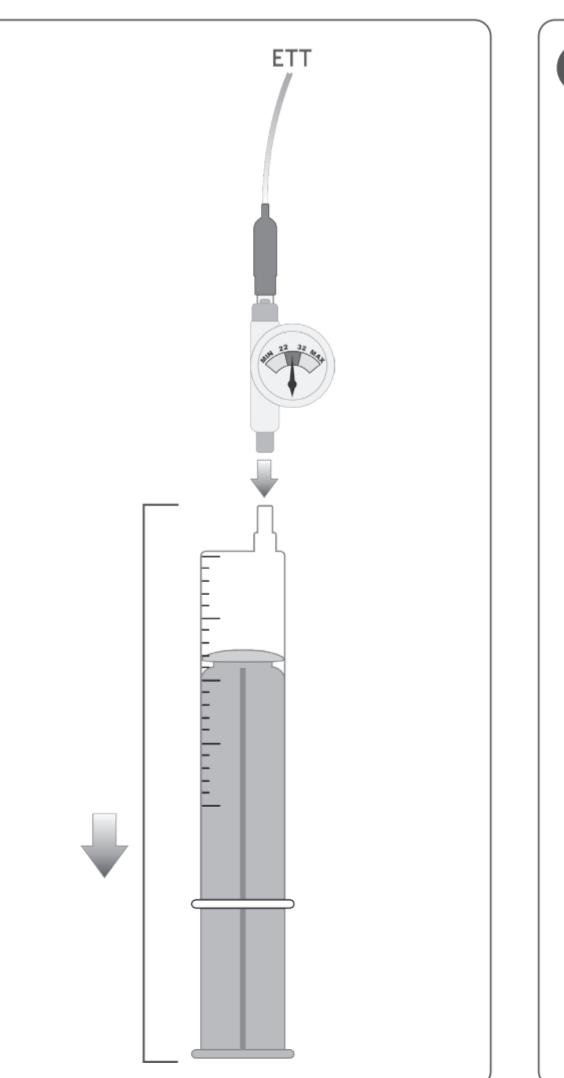
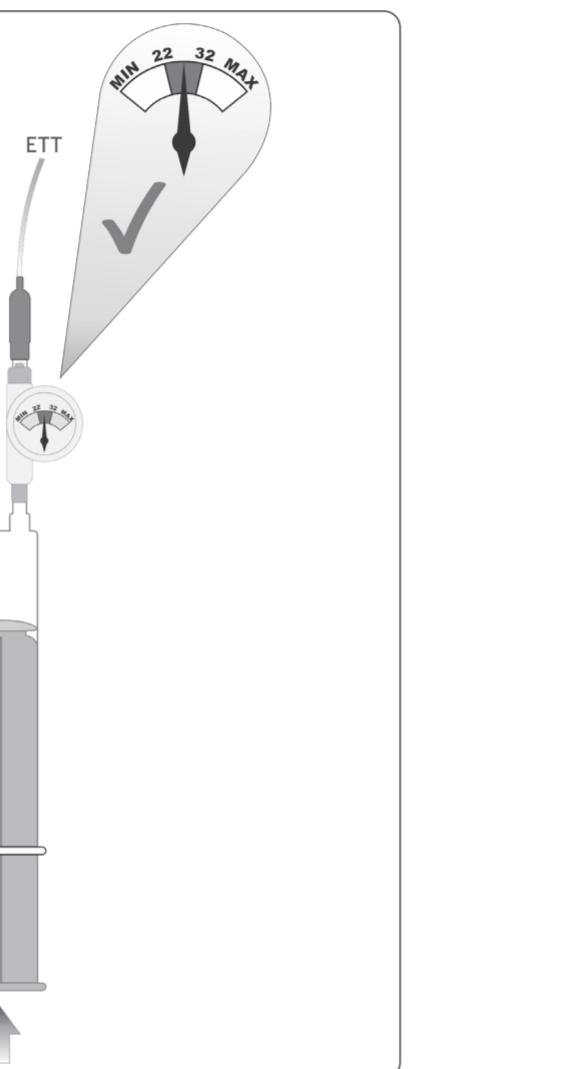
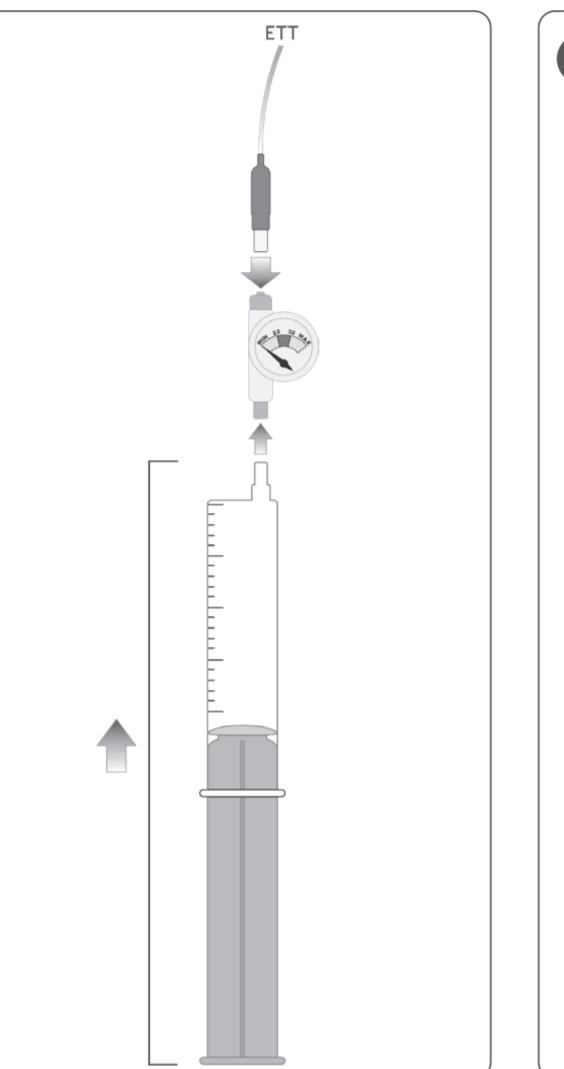
NOVO CP Cuff Pressure Monitor 30

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EN | Instructions for use

The NOVO CP Cuff Pressure Monitor 30 is an endotracheal tube (ETT) or tracheostomy tube cuff pressure manometer. It is used to indicate the pressure within the ETT cuff or tracheostomy tube cuff to prevent inadvertent over or under inflation of the endotracheal tube cuff.

1. Useful life

Replace if soiled / defective / according to hygiene guidelines.

2. Notice on reporting requirements

Please inform the manufacturer and the competent authority of the Member State in the event of any serious incident that has occurred in relation to the device.

3. Contraindications

N/A

4. Warning notices

May only be used by qualified clinical personnel.

The product is only suitable for use with endotracheal or tracheostomy tubes in adults.

Do not use the product with laryngeal masks.

Please wipe and disinfect the product with 75% alcohol gauze before use.

Not autoclavable. Not re-sterilisable

Do not use the product if the packaging has been damaged or stored improperly; it should be discarded.

It is strictly forbidden to inflate the product at high pressure. Frequent inflation with excessive pressure will result in the product no longer functioning properly or being damaged directly.

The product is a single-patient product and can be used multiple times on a single patient. Re-use on more than one patient may lead to cross-contamination.

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