INTERNAL USE ONLY
— DO NOT PRINT THIS PAGE —

ENGLISH

HEMCON®

HemCon® ChitoGauze® XR PRO

X-RAY
detectable

Intended Use / Indications:

HemCon ChitoGauzeXR PRO is a hemostatic dressing for the external, temporary control of severely bleeding wounds.

ChitoGauzeXR PRO Instructions for use:

1. Open foil pouch and remove ChitoGauzeXR PRO.
2. Identify and cover the source of bleeding with the dressing, apply direct digital pressure to the source of bleeding.
3. Pack dressing completely into wound track. Use enough ChitoGauzeXR PRO to fill the wound and contact all bleeding surfaces. More than one dressing may be required.
4. If cutting the dressing to a smaller desired size, ensure the radiopaque element is included in the parts you are using.
5. Apply pressure until bleeding is controlled.
6. Wrap and secure dressing on wound to maintain pressure.

FOR USE BY TRAINED EMERGENCY RESPONDERS

Cautions / Warnings:

FOR EXTERNAL USE ONLY, DO NOT IMPLANT.

Do not use if seal is broken and/or contents are wet. Do not apply over eyes. Do not use near the spine. Use promptly after opening. Contains chitosan from shellfish. Single use only. Discard any unused portion of the product. Do not resterilize. Not made with natural rubber latex.

Seek Medical Attention:

- If bleeding does not stop or redness or swelling occurs.
- If you suspect an allergic reaction.

Manufactured for:
 Tricol Biomedical, Inc.
 720 SW Washington Street, Suite 200
 Portland, OR 97205 U.S.A.
 www.tricolbiomedical.com | info@tricolbiomedical.com
 +1.503.245.0459 | Toll Free (U.S. & Canada) 877.247.0196
 [CE REP] :CEpartner4U, Esdoornlaan 13,
 3951DB Maarn, The Netherlands
 www.cepartner4u.eu

Distributed by:

SAFEGUARD MEDICAL™

Prometheus Medical Technologies
The Old Rectory, Hope Under Dinmore,
Herefordshire, HR6 0PW, UK
www.prometheusmed.com

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